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en english

These instructions for use can also be downloaded from:

www.bauerfeind.com/downloads

Dear Customer,

Thank you for choosing a Bauerfeind product that meets very high medical and quality standards.

Please read through the instructions for use carefully and in full, because they contain important information. Please keep these instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician, medical retailer or our customer service department.

Intended use

Orthopædic insoles are functional orthoses for the feet and are suitable for use in treating and preventing foot complaints.

Indications

These foot orthoses provide effective support for the treatment of foot deformities and complaints, such as *spring foot* /*arch decreased foot*. The treating physician is responsible for the precise diagnosis.

Risks of using this product

The foot orthosis is not suitable for people who have diabetes (diabetic foot syndrome) or rheumatism (rheumatoid foot deformities), as it does not provide soft enough cushioning for these types of foot complaints.

Before using / applying a foot orthosis, you should consult a physician and an orthotist who can help with the adjustment.

If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width for shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing).

If the feet are positioned on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive mechanical strain leads to holes or creases developing in the top covers, cushioning material or the sole of the shoe or to the breakage of these. You must stop wearing foot orthoses if they are damaged in this way!

Creases in stockings or pantyhose must be pulled flat before the shoes are put on, as creases in this kind can cause blisters and pressure points.

Kontraindikations

Diabetic foot syndrome and rheumatoid foot deformities.

Application instructions for the wearer of the foot orthoses

Shoes suitable for wearing with foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length /width, removable, sufficiently thick cushioned sole, adequate lateral guidance of the foot orthoses in the case of an open shoe fit).

An uncomfortable fit that constricts the foot must always be avoided.

When selecting the correct size of foot orthosis, the packaging information serves merely as a guide. Just like correctly fitting shoes, foot orthoses must be at least 1.0 cm longer than the foot, when the foot is in a weight-bearing, standing position. As feet may differ in length, please always check both feet and, if in doubt, base your size selection on the longer foot.

Foot orthoses must always be worn as a pair. Any exceptions to this must be stipulated by your physician.

The foot orthosis must be positioned straight in the shoe and have as much surface contact as possible with the shoe insole without slipping. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet.

If you are uncertain about anything during the fitting or regarding the fit, please contact your physician.

Please regularly check your feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact your orthotist or your physician.

Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to allow moisture to evaporate. The foot orthoses are not machine-washable. All surface materials facing the foot can be wiped if required, using a mild cleanser and an optical fibre cloth. After cleaning, the orthoses must be at least 1.0 cm longer than the wear. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight).

Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of the foot orthoses.

Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

Notes on reuse

The product is designed to provide care for an individual patient /individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper care of the product.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. Please directly contact the person who gave you the product in the event of a potential claim under the warranty.

Please do not make any changes to the product yourself! This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty. Discoloration in particular of the top covers of the foot orthoses may occur during their use. This is dependent on a number of factors, such as moisture levels and sock /stocking materials. This is not covered by warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

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