**de** **deutsch**

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## en english

## Dear Customer,

thank you for choosing a Bauerfeind product that meets very high medical and quality standards. Please read through the **instructions for use** carefully and in full, because they contain important information. Please keep these instructions for your safe, as you may want to read them again later. If you have any questions, please contact your physician, medical retailer or our customer service department.

### Product Description

The foot orthoses in the milled orthoses product group may be either non-prescription or prescription products that can be worn under the guidance of a physician in accordance with the specifications in these instructions for use and for the listed indications.

### Purpose

The foot orthoses in the milled orthoses product group are high-quality orthopedic aids used to prevent and provide conservative treatment of foot deformities.

### Indications/Areas of Application

- flat feet, excessive pronation, high arched/ cave feet, splay feet / metatarsalgia, and combinations of these deformities
- musculoskeletal problems related to foot loading or foot deformities, particularly knee- and hip-joint complaints and spine problems

### Risks of using this product

No side effects that affect the entire body have been reported to date. We require that trained specialists\* (orthotists) customize the foot orthoses in the milled orthoses product group for a proper fit that matches the indications.

### Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and hyperthermia.

- Sensory disorders in the relevant extremity
- Lymph drainage disorders, including unclear soft tissue swelling in the area where the medical aid will be used

### Application instructions

Product liability cannot be assumed in the event of inappropriate use.

- Your new pair of milled foot orthoses is a custom therapeutic device that has been customized for the problems and loading conditions affecting your feet by an Orthotist & Prosthetist or Certified Podiatrist.
- Only put your milled foot orthoses into shoes of the correct size and fit that can accept orthoses. If you have chosen milled sport-specific orthoses, please only wear them in the athletic footwear/sports equipment that is appropriate for the sport in question.
- We recommend that you get used to the foot orthoses gradually. If necessary, your orthotist can modify the orthoses.
- Leave your shoes with your orthotist to have your foot orthoses custom fitted. This is the only way to guarantee that your orthoses fit and function perfectly.
- Remove the insoles that come with your shoes/athletic footwear before you put your milled orthoses into the shoe. The thickness of the orthoses should not negatively affect the interior volume of the shoe that is available for the foot.
- Make sure that the foot orthoses lie straight in the shoe, without wrinkles, that your heel sits securely in the shoe and that the shoe can be laced properly.
- Foot orthoses come as a pair and may not be worn on one side only!

### Cleaning instructions

- Take insoles out of your shoes after wearing to air them out and to air-dry them. Do not expose the orthoses to a heat source, e.g., a heater!
- Foot orthoses covered with Alcantara® or a similar microfiber cover can be gently cleaned, unlike leather. Simply wipe the surface with a moist cloth. Do not wash the insoles in the washing machine.
- Your milled orthoses are therapeutic devices with a life span depending on usage. We recommend having your inserts inspected by your Orthotist & Prosthetist or Certified Podiatrist near six months after the last use.
- You can locate a specialist retailer (Orthotist & Prosthetist or Certified Podiatrist) near you by checking our list of retailers at **www.bauerfeind.com**.

We have inspected this product as part of our integrated quality control system. However, if you have any complaints, please contact your local specialist retailer or our customer service department.

### Notes on reuse

The product is designed to provide care for an individual patient/individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product.

### Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you make a claim that a product may be made under the warranty, please first contact the retailer from whom you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

### Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-01

\* A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic foot orthoses.

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