**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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Material
Polyamide (PA)
Polyurethane (PUR)
Spring Steel Strip, coated
Elastodiene (ED)
Viscose (CV)
Cotton (CO)
Thermoplastic Polyurethane (TPU)
Polyester (PES)
Elastane (EL)

(de) deutsch.....3	(sv) svenska.....10	(hu) magyar.....17	(lv) latviešu.....25
(en) english.....4	(no) norsk.....11	(hr) hrvatski.....19	(lt) lietuvių.....26
(fr) français.....5	(fi) suomi.....12	(sr) srpski.....20	(ja) 日本語.....27
(nl) nederlands.....6	(da) dansk.....13	(sl) slovenski.....21	(ko) 한국어.....28
(it) italiano.....7	(pl) polski.....14	(ro) romanian.....22	(ar) عربي.....30
(es) español.....8	(cs) česky.....15	(tr) türkçe.....23	(zh) 简体中文.....31
(pt) português.....9	(sk) slovensky.....16	(ru) русский.....24	

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en english

**These instructions for use can also be downloaded from:**  
[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)

## Dear customer,

Thank you for choosing a Bauerfeind product. Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

## Purpose

LumboLoc is an anatomically contoured lumbar support promoting muscle activation and relief. In addition, the specially positioned rods ensure anatomically correct support for the lumbosacral transition. The Velcro fastening allows easy closure, adjustment and adaptation to most requirements. LumboLoc employs the therapeutic principle of functional muscle activation. Therapy without side effects, avoiding the need for medication.

**LumboLoc must be adapted to each individual patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wearing comfort from this orthosis.**  
**If the product is not used properly any claim for warranty will expire.**

## Indications

- Nonspecific low back pain with weak trunk-supporting muscles
- Irritation of the small intervertebral joints (facet syndrome)
- Degenerative, painful spinal changes

## Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the LumboLoc. He will also inform you whether the LumboLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the LumboLoc as well as the instructions for use.

- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The product must be applied over a T-shirt or an undershirt.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Correct usage/fitting is assumed. Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the LumboLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.

**Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.**

## Contraindications

Side effects involving the body as a whole have not been reported to date.

In the following conditions such aids should only be fitted and applied after consultation with your medical advisor:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation.
3. Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is worn during severe physical exertion).

## Application instructions

### Fitting LumboLoc

- Position the support around the trunk, ensuring that the stays are located centrally at the back (Fig. 1).
- Place your fingers in the straps specially provided on the right and left hand fastenings and uniformly pull the support forwards (Fig. 2).
- Press the left hand fastenings against your abdomen and then pull the right hand fastenings until the two halves can be secured (Fig. 3).
- When securing the Velcro fastening withdraw your fingers from the left strap first, followed by the right strap (Fig. 4).

## Washing instructions and product care

**Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!**

Please refer to the care instructions on the sewn-in label of your LumboLoc. This can be found on the inner part of the knitted fabric.

1. Close the fastening of the orthosis.
2. Wash the product by hand.
3. The product is not suitable for tumble drying. Wash your orthosis regularly to retain the compressive effect.

To make a complaint, please contact the retailer that provided you with the product.

**Note:** Only clean goods are handled.

## Part of the body this product is used for

In accordance with the indications (lumbar spine). See intended use.

## Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

## Notes on reuse

This product is intended for your treatment only. Do not pass the orthosis on to other people.

## Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the LumboLoc in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

## Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Version: 2018-09

## Instructions for the orthopedic technician

If necessary the stays can be adjusted.

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