 **BAUERFEIND AG**  
Triebeser Straße 16  
07937 Zeulenroda-Triebes  
Germany  
P +49 (0) 36628 66-40 00  
F +49 (0) 36628 66-44 99  
E [info@bauerfeind.com](mailto:info@bauerfeind.com)  
**BAUERFEIND.COM**

## EpiPoint®

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**These instructions for use can also be downloaded from:**  
[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)

## Dear Customer,

thank you for choosing a product from Bauerfeind. EpiPoint is a high-quality medical product.

Please read through these **instructions for use** carefully and if you have any questions contact your doctor, medical retailer or our customer service department.

## Intended use

EpiPoint is a product for treating inflammation of the tendons on the outside of the elbow (epicondylitis). It works by applying pressure to the affected forearm muscle. The armband consists of a plastic liner with an integrated steel strap. Rotation of the adjustable pad enables the armband to be worn on both the right and left arms. The pressure of this right / left adjustable pad can be adjusted with the graduated Velcro® fastening.

## Indications

- Irritation of the tendon insertion, e.g. tennis elbow, golfer's elbow (epicondylopathy)
- Inflammation of the muscle origin (chronic or recurring)

## Risks of using this product

- **EpiPoint is a prescribed product that should be worn under a physician's guidance. EpiPoint should only be applied in accordance with these instructions for use and for the listed areas of application.**
- Should you notice any unusual changes or increased symptoms, please contact your physician without delay.
- No product liability is accepted in the event of improper use.

- The EpiPoint should not be altered arbitrarily.
- In the event of a product defect or wear to individual parts, please contact your medical retailer.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- The product's unique effect is produced primarily during physical activity. The product should be removed during prolonged periods of rest (e.g. sleeping, continuous sitting).

 Contains natural rubber latex (elastodiène), which can cause skin irritation in persons with an allergy. Take off the orthosis if you notice any allergic reaction.

## Contraindications

Side effects involving the body as a whole, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the forearm and the hand.
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin located remotely from the fitted aid.
4. Bottleneck syndromes (irritation due to compression of nerves in the arm).

## Application instructions

### Putting on the EpiPoint

1. Undo the band and place the EpiPoint strap on the forearm, ensuring the five-point pad rests directly on the muscle belly, i.e. lateral position (outside) in case of tennis elbow or medial position (inside) in case of golfer's elbow. The rigid part of the strap should always face outwards. Position the pad approx. 4–6 cm below the elbow.
2. Adjust the Velcro® fastening for the largest circumference and secure. Slip the armband onto the affected forearm so that the rigid portion of the armband always faces outwards.

### Product care and washing instructions

Please refer to the care instructions on the sewn-in label of your EpiPoint.

EpiPoint can be washed separately by hand at 30 °C using a mild detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaints, please contact your retailer or our customer service department.

### Part of the body this product is used for

In accordance with the indications (elbow). See purpose.

### Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

### Information about re-use

The EpiPoint is intended for your treatment only. It has been adapted

specifically for you. For this reason, do not pass the EpiPoint on to other people.

### Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

### Disposal

You can dispose of the product in accordance with national legal requirements.

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Bauerfeind Middle East FZ-LLC
Bauerfeind USA, Inc.