AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear client,

thank you for choosing a product from Bauerfeind. AchilloTrain Pro is a high-quality medical product. Please read through these **instructions for use** carefully. If you have any questions please contact your doctor, your local retailer or our Technical Service department.

Intended use

AchilloTrain Pro, the active Achilles tendon support is anatomically knitted which ensures an excellent fit. The massage effect of the shaped viscoelastic insert around the Achilles tendon area promotes lymph drainage and circulation and helps reduce swelling and bruising. The stretchable knitted section around the shin also facilitates donning of the support.

⚠ AchilloTrain Pro must be fitted for the first use by a trained specialist*, otherwise we cannot accept any liability for the product.

Indications

- Painful irritation of the Achilles tendon (achillodynia) and adjacent tissues
- After surgery, e.g. tears in Achilles tendon

Risks of using this product

⚠ Important Information

- AchilloTrain Pro is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her/him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your doctor.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to

the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves

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Contra-indications

Hypersensitive reactions harmful to health have not been reported to date.

In the following conditions such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the feet, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting AchilloTrain Pro

1. Take hold of the upper edge of the support and pull it over the foot until the heel reaches the blue knitted section **1** + **2**.
2. Ensure that the Achilles tendon is securely held by the shaped insert.
3. Adjust the fit of the support as necessary.

Washing instructions and product care

⚠ All Train products can be machine washed separately at 30 °C with liquid detergent². Please note the following when washing your Train products:

1. For products with Velcro straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads / inserts and their terrycloth covers before washing.

We recommend using a mesh bag. Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric. Please wash your Train product regularly so that the compression of the knitted fabric can be retained. This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Notes on reuse

The product is intended to be fitted only once and to one patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

The product can be disposed of without any restrictions.

Version: 2018-01

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

* A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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