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en — english —

These instructions for use can also be downloaded from: **www.bauerfeind.com/downloads**

Dear Customer

Thank you for choosing a Bauerfeind product that meets very high medical and quality standards. Please read through the instructions for use carefully and in full, because they contain important information. Please keep these instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician, medical retailer or our customer service department.

Intended use

Orthopedic orthoses are functional orthoses for the feet and are suitable for use in treating and preventing foot complaints.

Indications

These foot orthoses provide effective support for the treatment of foot deformities and complaints, such as skew foot / arch decreased foot in children. The treating physician is responsible for the precise diagnosis.

Risks of using this product

Before using / applying a foot orthosis, you should consult a physician or specialist* on the risks mentioned above.

If there is not enough space inside the shoe for the foot and the selected foot orthosis, or if the foot orthoses are worn in shoes of an incorrect length and width/ for shoes with inadequate lateral guidance in the case of open shoe types), problems relating to poor fit may arise (e.g. a feeling of constriction or pressure, rubbing).

If the feet are positioned on the edges of the foot orthoses, blisters and calluses may form on the soles of the feet. The same symptoms may occur if excessive mechanical strain leads to holes or creases developing in the top covers, cushioning material or orthotic cores, or to the breakage of the sole. You must stop wearing your foot orthoses if they are damaged in this way!

Creases in stockings or pantyhoses must be pulled flat before the shoes are put on, as creases of this kind can cause blisters and pressure points.

Contraindications

The foot orthosis is not suitable for people who have diabetes (diabetic foot syndrome) or rheumatoid arthritis (rheumatoid deformities), as it does not provide soft enough cushioning for these types of foot complaints.

Application instructions for the wearer of the foot orthosis

The foot orthosis is exclusively intended for use by growing children following clinical assessment by a specialist.

Combining the aid with other medical products (e.g. compression stockings, supports) may only be considered after having sought specialist advice.

Shoes suitable for wearing with foot orthoses must guarantee sufficient internal volume for both the foot and the selected foot orthosis (adequate shoe length/width, removable, sufficiently thick cushioned sole, adequate lateral guidance of the foot orthoses in the case of open shoe types). An uncomfortable fit that constricts the foot must always be avoided. **!**

When selecting the correct size of foot orthoses, the packaging information serves only as a guide. Please check the fit of the foot orthosis. If you are unsure which size to choose, please contact your specialist.

The foot orthosis should always lie flat at the midfoot (b1) and hindfoot (b2) – without tilting. **!** Foot orthoses must always be worn as a pair! Any exceptions to this must be stipulated by your physician.

The foot orthosis should be positioned straight in the shoe and have as much surface contact as possible with the shoe insole. Contact between the foot orthosis and the shoe upper helps reduce the risk of pressure marks or blistering on the skin of the soles of the feet. **!**

If you are unsure about anything during the fitting or regarding the fit, please contact your local specialist.

Please regularly check your child's feet for any pressure marks, redness or other changes caused by the foot orthosis. If you have any problems, please contact a specialist or your physician.

Adjustment phase

As with many medical products, foot orthoses may feel somewhat strange to wear at first. Where therefore recommend that your child starts by wearing the foot orthosis for 1–2 hours at a time and slowly increases the wearing period until it feels comfortable. Normally, it should not take longer than a week to adjust to wearing the foot orthosis. If your child still finds it uncomfortable to wear after this period, please contact a specialist.

The aid supports your child while they are growing and therefore requires particular attention. Please ask your specialist about appropriate check-up intervals.

Cleaning instructions

Foot orthoses must be removed from the shoes on a daily basis in order to allow moisture to evaporate. The foot orthoses are not machine-washable.

All surface materials facing the foot can be wiped if required, using a mild cleanser and a sponge or cloth. After cleaning, any residual soap must be thoroughly rinsed off with water. The orthoses must be left to air dry and no external heat source should be used (heating, hair-dryer, direct sunlight).

Mild disinfectants suitable for human skin may be applied sparingly (sprayed on). However, this may lead to discoloration of top cover materials.

Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected foot orthosis.

Notes on reuse

The product is designed to provide care for an individual patient / individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product.

Warranty

Foot orthoses are designed for daily use and as such they are subject to natural wear when used on a regular basis.

Signs of deterioration, in particular fading of the top covers of the foot orthoses, may occur during their use. This is dependent on a number of factors, such as moisture levels and sock / stocking materials. This is not covered by warranty. The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply here. If there is a claim or a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposeal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-12

* A specialist is any person who is authorized according to the state regulations that are relevant to you for fitting and instruction in the use of orthopedic orthoses.

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Bauerfeind Benelux B.V.