AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 WIEN
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
info@bauerfeind.at

BENELUX

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en english

Dear patient,

Thank you for choosing a product from Bauerfeind. SecuTec Dorso is a high-quality medical product.

Please read the **instructions** carefully and if you have any questions contact your medical advisor, medical retailer or our Technical Service department.

Purpose

SecuTec Dorso is a modified stay-based/Lindemann orthosis for the support of the lumbar spine up to, and including, the lower thoracic spine. The support also relieves the lumbar spine by applying graduated compression to the soft tissues of the abdomen.

⚠ SecuTec Dorso must be individually adapted to the patient by a trained orthotist in order to gain the maximum effect and optimal wear comfort from this orthosis. If the product is not used properly any claim for warranty will expire.

Indications

- Bone atrophy (osteoporosis)
- Sliding vertebrae (spondylolisthesis)
- Degenerative instabilities
- Postsurgical stabilization of the lumbar spine

Risks of using this product

⚠ Important information

- SecuTec Dorso is a prescription product that should be applied under a medical provider's guidance. SecuTec Dorso should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product should be applied over a T-shirt or an undershirt.
- Incorrect use of the support will void any product liability.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical advisor without delay.
- Please consult your medical advisor before using SecuTec Dorso in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body may, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

LATEX Contains natural rubber latex (elastodiène), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contra indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- 1 Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- 2 Impaired sensation.

- 3 Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is applied during severe physical exertion).

Application instructions

Notes for the orthotist

If necessary, the corset stays may be adapted.

Fitting and removing SecuTec Dorso

Fitting:

- Place the orthosis around the trunk so that the stays interlinked in the grid are positioned across the back, as shown in Fig. 1 (sewn-in labels at the top).
- Place the fingers in the specially provided finger pockets on the two broad, light-gray fastener halves and pull forward uniformly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the Velcro fastening (Fig. 2).
- When securing the Velcro fastening, remove the fingers from the left finger pocket first, followed by the right (Fig. 3).
- Take the two lower LPT (Low Power Tension) straps in both hands and pull forward at the same time to produce the desired level of force and secure both LPT straps against the large Velcro fastening (Fig. 4). Repeat this process for the upper LPT tension straps (Fig. 4) until the orthosis is correctly fastened (Fig. 5).
- Correct fitting should be practiced with the patient.

Removal:

- Unfasten the LPT tension straps (first the upper followed by the lower straps) and then refasten without any tension.
- Open the large, light-gray fastener.
- Remove the orthosis and refasten the light-gray fastener.
- For easier handling of SecuTec Dorso, always fasten any opened fasteners directly on the product.

Cleaning instructions

- We recommend washing SecuTec Dorso separately by hand with a mild detergent at 30° C (86°F). To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).
- Removal of the corset stays is not necessary.
- If the stays are removed, turn back the fasteners at the top of the channels for the corset stays and pull out the stays.
- When reinserting the corset stays after washing, ensure that the stays are inserted in the correct pockets. This can readily be determined by pocket size. Ensure that the corset stays are completely covered by the receiving channels and that the channels are refastened.
- During insertion of the stays, the marking must be at the top and face outwards (away from the body).
- Do not apply excessive force to the stays as this can alter the anatomical contouring of the stays.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your medical retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications. See Purpose.

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly and fitting instructions

The SecuTec Dorso is supplied in a standard size and ready for use. The bridging frame must be individually adapted by a trained specialist and positioned in the knitted section.

Technical specifications / parameters, accessories

The SecuTec Dorso is an orthosis for the back. It consists of a knitted section, a velour pocket for inserting the bridging frame, an abdominal pad and four double-sided micro Velcro fastenings. The velour pocket and bridging frame are available as spare parts on request.

Notes on reuse

SecuTec Dorso is intended for single use by the patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

SecuTec Dorso can readily be disposed of with the domestic refuse.

Version: 2016-07

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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