Material

Silikon (SI)

Material

Silicone (SI)

 **BAUERFEIND AG**

Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-35 00
F +49 (0) 36628 66-39 99
E info@bauerfeind.com

BAUERFEIND.COM

ViscoHeel® K

Viskoelastische Einlagen
Viscoelastic heel cushions
Talonnettes visco-élastiques
Taloneras viscoelásticas

(de)	deutsch2	(mk)	македонски23
(en)	english3	(sr)	srpski24
(fr)	français4	(sl)	slovenski25
(nl)	nederlands6	(ro)	romanian26
(it)	italiano7	(el)	ελληνικά28
(es)	español8	(tr)	türkçe29
(pt)	português10	(ru)	русский30
(sv)	svenska11	(et)	eesti32
(no)	norsk12	(lv)	latviešu33
(fi)	suomi14	(lt)	lietuvių34
(da)	dansk15	(uk)	українська35
(pl)	polski16	(he)	עברית37
(cs)	česky17	(ja)	日本語38
(sk)	slovensky19	(ko)	한국어39
(hu)	magyar20	(ar)	عربي41
(hr)	hrvatski21	(zh)	简体中文42

(de) deutsch

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(en) english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear Customer,

Thank you for choosing a product from Bauerfeind. The ViscoHeel K you have purchased is a high-quality medical product.

Please read through the **instructions for use** carefully, because they contain important information.

Please keep the instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

Intended use

ViscoHeel K are soft, viscoelastic heel cushions with a surface that slopes downward at the side. ViscoHeel K heel cushions do not slip and can be worn in everyday shoes, as well as in sports shoes and work shoes.

Indications

- For varus and valgus misalignment of the heel

Risks of using this product

⚠ Caution*

ViscoHeel K must only be worn in accordance with the specifications in these instructions for use. To ensure the optimum fit of the ViscoHeel K, the size required must be determined exactly. Only a trained specialist¹ may carry out the (initial) fitting of the product and provide instruction.

Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids. No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

Skin disorders /injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

Application instructions

Inserting the ViscoHeel K

Use the ViscoHeel K in shoes with a stiff heel counter. Discuss the use of the ViscoHeel K with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

The ViscoHeel K heel cushions are used to correct varus or valgus misalignments of the heel. The heel cushions are marked with the letters A and B.

- To correct a varus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the right shoe, heel cushion B in the left shoe.
- To correct a valgus misalignment of the heel, insert the heel cushions as follows: heel cushion A in the left shoe, heel cushion B in the right shoe.

Unless otherwise prescribed by your physician, place **both** ViscoHeel K heel cushions in your shoes so they fit snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

Cleaning instructions

Note: Never expose the ViscoHeel K to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the ViscoHeel K.

If necessary, you can clean your ViscoHeel K heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

Notes on reuse

ViscoHeel K is intended for your treatment only and has been specifically selected according to your measurements. For this reason, do not pass the ViscoHeel K on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our

instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Version: 2017-03

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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Bauerfeind Middle East FZ-LLC
Bauerfeind USA, Inc.