 **BAUERFEIND AG**  
Triebeser Straße 16  
07937 Zeutenroda-Triebes  
GERMANY  
P +49 (0) 36628 66-40 00  
F +49 (0) 36628 66-44 99  
E info@bauerfeind.com

## ViscoSpot®

Viscoelastische Einlagen  
Viscoelastic heel cushions  
Talonnettes visco-élastiques  
Taloneras viscoelásticas

 **MADE  
IN  
GERMANY**

de	deutsch	2	hr	hrvatski	21
en	english	3	sr	srpski	22
fr	français	4	sl	slovenski	23
nl	nederlands	6	ro	romanian	24
it	italiano	7	el	ελληνικά	26
es	español	8	tr	türkçe	27
pt	português	9	ru	русский	28
sv	svenska	11	lv	latviešu	30
no	norsk	12	li	lietuvių	31
fi	suomi	13	uk	українська	32
da	dansk	14	he	עברית	34
pl	polski	15	ja	日本語	35
cs	česky	17	ko	한국어	36
sk	slovensky	18	ar	عربي	38
hu	magyar	19	zh	简体中文	39

de deutsch

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en english

## Dear Customer,

Thank you for choosing a product from Bauerfeind. The ViscoSpot you have purchased is a high-quality medical product.

Please read through all of these instructions for use carefully, because they contain important information. Please keep these instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

### Intended use

ViscoSpot is a viscoelastic heel cushion with softer anatomically designed pressure relief zones (blue and white area). ViscoSpot heel cushions do not slip and can be worn in everyday shoes, as well as in sports shoes and work shoes.

If the product is not used properly, all warranty rights will be rendered null and void.

### Indications

- Heel pain and talagia in cases of tendon attachment site irritation ("calcaneal spurs")
- Wear to the leg and vertebral joints (shock absorption)
- Knee and hip prostheses (shock absorption)
- Irritation of the Achilles tendon and the surrounding tissue
- Hindfoot pain

### Risks of using this product

#### ⚠️ Caution\*

- ViscoSpot must only be worn according to the specifications in these instructions for use. To ensure the optimum fit of the ViscoSpot, the size required must be determined exactly.
- Only a trained specialist<sup>1</sup> may carry out the (initial) fitting of the support and provide instruction.
- Combination with other products, e.g. for evening out leg lengths, should be discussed beforehand with the physician treating you.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.
- No product liability is accepted in the event of improper use.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.

- Side effects that affect the entire body have not been reported to date. Correct fitting is assumed.

## Contraindications

No clinically relevant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and applied after consultation with your physician:

- Skin disorders/injuries in the respective part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.

## Application instructions

### Inserting the ViscoSpot

Use the ViscoSpot in shoes with a stiff heel counter. Discuss the use of the ViscoSpot with the physician treating you beforehand, particularly if you want to use it in cushioned running shoes.

Unless otherwise prescribed by your physician, place both ViscoSpot heel cushions in your shoes so the shape fits snugly into the heel area of the shoes. Please take the right and left versions into account when doing this.

## Cleaning instructions

Note: Never expose the ViscoSpot to direct heat (e.g. heater, sunlight, in your car, etc!.) This may cause damage to the material, which can impair the effectiveness of the ViscoSpot.

If necessary, you can clean your ViscoSpot heel cushions with a mild detergent in lukewarm water. Leave the heel cushions to air dry.

## Notes on reuse

The ViscoSpot is intended for your treatment only and has been specifically selected according to your measurements. For this reason, do not pass the ViscoSpot on to other people.

## Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our

instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

## Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2016-10

\* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

<sup>1</sup> A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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Bauerfeind Middle East FZ-LLC
Bauerfeind USA, Inc.