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These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear Patient,

Donning and removing your compression stockings is extremely comfortable with the VenoTrain glider. Your leg or arm slips in the compression stocking without any effort. The VenoTrain glider also helps you remove your stockings. Ease of handling is not the only benefit of the donning and removal aid.

Cleaning Instructions

The VenoTrain glider is hard-wearing and absolutely easy to clean. Just wipe it with a moist cloth – that is all.

Folded up and tucked away in the storage bag, your light-weight VenoTrain glider fits any carrying bag.

The way how to use the VenoTrain glider is illustrated on the following pages.

Indications

- Restricted mobility

Application Instructions

Donning aid (legs)

- Put the foam surface – point forward – on the floor in front of you.
- Then place the VenoTrain glider on with the white side up.
- Put the tip of your foot in the marked semicircle and push it forward with slight pressure.
- Your foot glides into the VenoTrain glider as if on its own accord.
- You can don your compression stocking as usual starting with the toes simply over your VenoTrain glider which allows you to don the stocking without trouble.
- With light movements of the tip of your foot, pull the VenoTrain glider out of the stocking at the tape at the rear – very simple. Please turn the VenoTrain glider to the correct side after using it (loop to point outward).
- If you wear compression stockings with open toe, pull the VenoTrain glider out of the stocking forward by the loop.

Removing aid (leg)

- Put the pull tape in the VenoTrain glider and slide in your foot. The blue side of the donning aid is up.
- Fold up your compression stocking to the VenoTrain glider and place the edge or stay-up tape over it.
- Pull the compression stocking over the heel and instep towards the tip of your foot. When you remove the stocking, your VenoTrain glider is inside it.

Donning aid (arm)

- Put the pull tape in the VenoTrain glider and slide your arm in it. The blue side of the donning aid is up.
- You can don your compression stocking as usual starting with the hand over your VenoTrain glider to the upper arm. Adjust your stocking on the arm.
- Attach the loop of the VenoTrain glider at a safe point and simply pull it out while you hold the compression stocking with your free arm.

Our Quality Assurance

This product was thoroughly checked before packaging. Should you have any complaints, please bring the product back to your specialist shop.

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Best Stored
Your VenoTrain glider is best protected stored in the attached nylon-bag.

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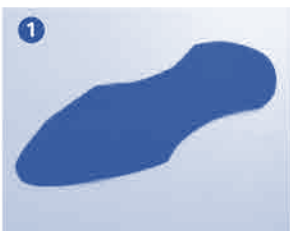
All instructions for use:
www.bauerfeind.com/downloads.com

BAUERFEIND AG
Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628 66-40 00
F +49 (0) 36628 66-44 99
E info@bauerfeind.com
BAUERFEIND.COM

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Bauerfeind USA, Inc.