AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear patient,

Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us.

Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

SacroLoc is a medical device. It is an orthosis¹ for the stabilization of the pelvis and targeted relief of the sacroiliac joints.

Indications

- Sacroiliac joint disorders (SI joint syndrome, SI joint arthrosis, SI joint instability)
- Pelvic girdle instability, e.g. symphysis rupture and loosening
- Sacroiliac joint blockage (acute and prophylactic)
- Myalgia and tendinopathy in the pelvic region
- Structural disturbances following lumbar spinal fusion (in the L5 – S1 transition region)

- Non-surgical treatment following pelvic fractures

Risks of using this product

Caution*

- SacroLoc must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application (places of use) listed.
- No product liability is accepted in the event of improper use.
- The suitability and use of the orthosis¹ during pregnancy is to be determined by a physician. If wearing the orthosis¹ during pregnancy feels uncomfortable, check size adjustments and application technique, seek medical advice, and refrain from further use if necessary. Side effects for the mother and fetus are not known.
- Only a professional² may provide instruction.
- Use in combination with other products, e.g. as part of compression treatment (compression stockings), must be discussed beforehand with your physician.
- The SacroLoc is to be worn under your T-shirt or undershirt.
- If you notice any unusual changes on your body or your symptoms get worse, take the product off and please contact your physician without delay. Please doff the orthosis during longer rest phases.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.
- Side effects involving the body as a whole have not been reported to date. Correct donning of the product is assumed. Do not don the SacroLoc too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. For this reason, check the fit of the aid if you feel unpleasant pressure and consult the professional in this regard or for general questions about handling.
- Caution: when loosening the straps or tensioning and fastening systems, or when doffing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.

Use during pregnancy

The fundamental suitability and use of the orthosis¹ during and after pregnancy is to be determined by a physician. If you should experience any discomfort while wearing the orthosis¹ during pregnancy, immediately doff the orthosis and consult your physician without delay. They will decide on whether and how you should continue using the product. Size adjustment and donning technique should be practiced and reviewed regularly with a professional.

Contraindications

No clinically significant hypersensitivity reactions have been reported to date. If you have any of the following conditions, such aids should only be donned and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Impaired sensation and movement disorders, e.g. in the case of diabetes mellitus

Application instructions

- A – Back pad (massage cushion)
- B – Right half of the fastening with finger loop
- C – Left half of the fastening with finger loop
- D – Tensioning straps

Donning

- 1 Position the orthosis¹ centrally on the pelvis, ensuring that the upper edge of the orthosis¹ is no higher than the upper edge of the iliac crest and below the waist. 2 Then slide your fingers downward into the finger pockets provided on the fastening and steadily pull the fastening forward. 3 Then press the left half of the fastening against the abdomen and position the right half of the fastening on top of the left half so that it can be fastened to it. When fastening the two halves, remove your fingers from the left-hand finger pocket first and then from the right. 4 Then take hold of the two tensioning straps and pull them forward, steadily and at the same time, until the desired level of tension (stabilization) is achieved. 5 Then fasten the two tensioning straps close together on the large fastening.

Cleaning instructions

Note: Never expose the SacroLoc to direct heat (e.g. heater, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the SacroLoc.

Please refer to the care instructions on the sewn-in label of your SacroLoc. This can be found on the inner part of the knitted fabric. If necessary, you can wash the SacroLoc by hand at 30 °C with a mild detergent. Remove the pads and Velcro patches beforehand. Leave the orthosis to air dry. The pads can be washed by hand with the Velcro patches removed at 30 °C using a mild detergent. Leave the pads to air dry.

Part of the body this product is used for

In accordance with the indications (pelvis, sacroiliac joints). Please see intended purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The SacroLoc is supplied in a standard size and ready for use.

Technical specifications / parameters

SacroLoc is made of a knitted fabric with firm tensioning straps and removable back pads. The pack pads are attached with an unremovable Terrycloth cover. The back pads can be supplied as spare parts.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. The warranty may be limited or excluded if the instructions on how to handle and care for the SacroLoc have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional^[2]. Before first using our medical device, please seek the advice of a physician or professional^[2] as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

Professionals²

General instructions

Check the optimal and individual fit of the SacroLoc after the first fitting. Adjust the position of the back pads against the pelvis so that the pads are lying with the inner recess at the posterior superior iliac spine. Practice the correct procedure for donning of the product with the patient.

No improper modifications may be made to the product. Failure to comply may adversely affect the product’s performance, thus canceling any product liability.

Material content

Polyamide (PA), Thermoplastic elastomer (SEBS), Polyurethane (PUR), Elastane (EL), Polypropylene (PP), Polyester (PES),

Polyoxymethylene (POM), Cotton (CO), Polyethylene (PE), Acrylate adhesive (A)

MD[ⓘ] – Medical device

UDI[ⓘ] – DataMatrix Unique Device Identifier UDI

Date of information: 2025-06

^{*}Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

²A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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