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Dear patient,  
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your physician or medical retailer / dealer.

### Intended use

ManuTrain is a medical device. It is a support for the relief and stabilization of the wrist.

### Indications

- Chronic, post-traumatic or post-operative irritation
- Distortions (wrist injuries e.g. sprains)
- Osteoarthritis (joint wear)
- Instability, e.g. radioulnar subluxation
- Tendovaginitis (tenosynovitis)
- (Relapse) prevention

### Risks of using this product

The benefits of this product are felt particularly during physical activity.

- Remove the support during longer periods of rest.
- Once you have been prescribed the ManuTrain, only use it according to your indication and any additional instructions given by a specialist\*. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves. Avoid fastening the strap too tight.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to grip is also heavily impacted.

### Contraindications

No clinically significant adverse reactions have been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation and circulatory disorders of the hand.
- Impaired lymph drainage and soft tissue swellings of uncertain origin including those located away from the fitted aid.

### Application instructions Putting on the ManuTrain

As an alternative to the following description, a donning video is available to you on the product details page. To access this, scan the QR code next to Figure 7 and follow the instructions in the video .

- Pull the support over the wrist until the joint is enclosed by the knit fabric and pad without any pressure being exerted on it.
- The support fits correctly when the bony protrusion on the outside of the wrist is enclosed by the pad's ring-shaped cutout ①.
- Fasten the narrow side of the strap (light blue velour) on the light blue Velcro area (if not already pre-fastened) ②. For optimum stabilization, move your wrist towards the back of your hand.
- Guide the strap to in the direction of your little finger ③, then over the palm of your hand between the thumb and index finger ④ and back to the wrist ⑤.
- Then wrap the strap around the wrist and fasten it to the wide strap ⑥ ⑦.
- Additionally, you can use the markings on the support to guide you (grey lines).
- For increased stabilization, loosen the strap and re-tighten it applying more tension.

### Alternative

- For more gentle treatment, separate the strap halves by separating the Velcro areas from one another (see Figures ①–⑭). Now fasten the Velcro area onto the identical arrow-shaped area on your support ① ② and guide the strap in the direction of the arrow (towards the outside of your wrist) around the wrist ③. Then fasten the strap in the desired position ④.
- In order to reconnect the strap halves, place both parts with the Velcro area facing upwards and outwards (see full strap figure in the image series ①–⑭). Follow the steps in reverse order from Figure ⑭ to ①.

### Removing the ManuTrain

- Loosen the Velcro fastening and attach the end of the strap onto the strap's Velcro to avoid damage to the support. Then loosen the strap and take off the support.  
**Make sure not to turn the support inside-out.**

### Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.  
For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.

### Notes on reuse

This product is intended for your treatment only.

### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ManuTrain have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

### Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a specialist\*. Before first using our medical device, please seek the advice of a physician or specialist\* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

### Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

### Material content:

Polyamide (PA), Styrene-ethylene / butylene-styrene block copolymer (TPS-SEBS), Spandex (EL), Polyurethane (PUR), Polyethylene (PE), Cotton (CO), Polyester (PES), UV stabilizer

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

\* A specialist is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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